

ALL HIGH-GRADE



MeGUARD PRO

HIGH QUALITY FACE GUARD FOR PROFESSIONALS

A special MeGUARD for the evolving frontline of medical care.

Recognized by professionals in hard medical care settings- High-quality face guard [MeGUARD]

[MeGUARD] is now available in a "top-of-the-line" model with even better texture, design, comfort, and durability.

We support higher quality medical care with quality that is one step ahead of the rest.



A special MeGUARD for the evolving frontline of medical care.

MeGUARD PRO

HIGH QUALITY FACE GUARD FOR PROFESSIONALS

How to attach
Click here to view.



Replaceable film type

Disposable type to ensure cleanliness and comfort at all times.

Can be lifted up

If the film gets in the way, it can be lifted it up.


Can be worn with glasses

The headband-style design places no burden on the nose, even when worn with glasses.


Easy replacement

The film is very easy to put on and take off and can be easily adjusted.


Please use the MeGUARD PRO main unit with the dedicated film attached.

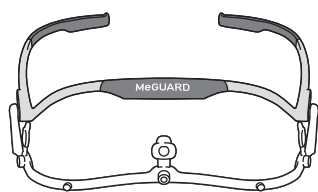

MeGUARD PRO main unit

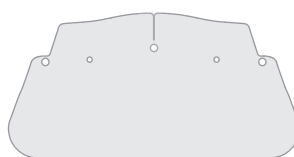


MeGUARD dedicated film
(sold separately)

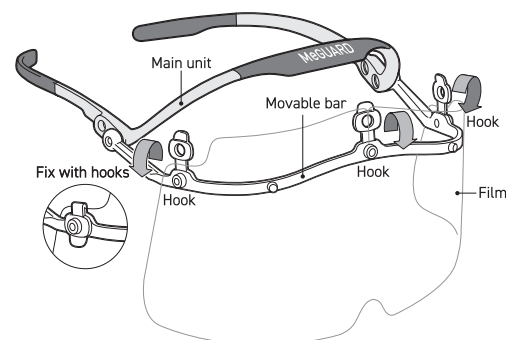


The position of the movable bar can be adjusted.

- MeGUARD PRO is connected by means of the main unit's holes and the movable bar's pins and screws.
- To take off, remove the screws with a screwdriver and then gently pull apart by holding the joint section on both sides.
- To attach, choose the conveniently positioned holes on the left and right sides of the main unit, push the movable bar pins straight into the main unit holes, and tighten the screws with a screwdriver.
- Caution: When attaching, be sure to align the holes on the left and right.

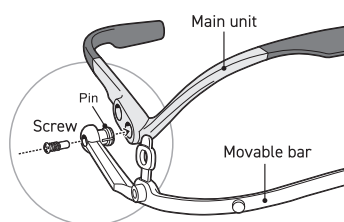

Please use with the film correctly attached.

- Attach the main unit so that the front part presses down on the top of the forehead.
- If you don't need the film, you can lift it up.

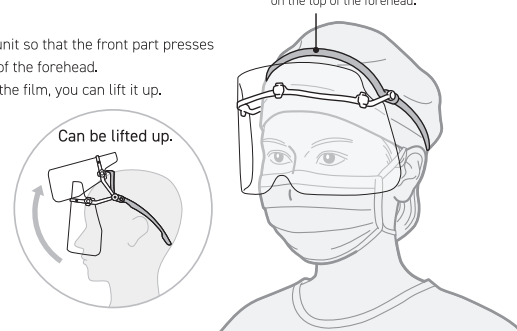

⚠ Precautions in use

- (1) The anti-fog film (MEG-1F) has a protective sheet on one side. When attaching the film to the main unit, please do so with the protective sheet still attached. Also, be sure to remove the protective sheet before use.
* This film is disposable. Avoid using a dirty film or using the same film multiple times. Replace with a new film each time you use.
- (2) Users should not modify or repair the product by themselves.

⚠ Precautions when handling

- (1) To clean the frame, hand-wash with a neutral detergent diluted in warm water, and wipe with a soft cloth or soft paper after rinsing.
- (2) To sterilize the frame, use anhydrous ethanol or a solution of sodium hypochlorite diluted to about 6%. Do not use gas, heat (boiling), organic solvents, or other incompatible chemicals, as they may cause deterioration or damage to the product.

⚠ This product is not sterilized.

MeGUARD PRO Main Unit Product Environmental and Safety Information

[Hazard Information] This product is not harmful to health if handled normally (originally intended conditions of use).

[Disposal Precautions] Dispose of the product in accordance with the Waste Disposal and Public Cleaning Law and local government ordinances. If incinerating, do so in accordance with the Waste Disposal and Public Cleaning Law, Air Pollution Control Law, and Water Pollution Control Law.

[Storage Precautions] Store in a recommended (temperature 15°C–20°C, humidity 45%–55%) location away from direct sunlight, and do not expose the product or packaging to water, solvents, or gases.

MeGUARD PRO Main Unit Specifications

- ☐ MeGUARD PRO Main unit (excluding screws, decorations and resin parts): Titanium
- ☐ Decorative parts: Alloy, resin [resin parts heat resistant up to approx. 70°C]
- ☐ Movable bar: Polypropylene [heat resistant up to approx. 90°C]
- ☐ Weight: approx. 33.0g (with M film), approx. 36.5g (with L film)

Film Specifications

- ☐ MEG-1F (M) [Half face]: w247mm x h123mm x t0.13mm
- ☐ MEG-1F (L) [Full face]: w248mm x h209mm x t0.13mm
- ☐ MEG-SC (M) [Half face]: w247mm x h123mm x t0.16mm

Product name	Item number	Packaging unit
MeGUARD PRO (Midnight)	MEGP-MB	1 pc
MeGUARD PRO (Cardinal Red)	MEGP-CR	1 pc
MeGUARD PRO (Titanium Gray)	MEGP-TG	1 pc

MeGUARD dedicated replaceable double-coated anti-fogging film M size	MEG-1F(M)	10 pcs per pack
MeGUARD dedicated replaceable double-coated anti-fogging film L size	MEG-1F(L)	10 pcs per pack
MeGUARD dedicated replaceable film (Super Clear) M size	MEG-SC(M)	10 pcs per pack

*Product specifications are subject to change without notice.

MADE IN JAPAN